# XCP-DS

## Digital Sensor Positioning System

### Placing the sensor in the holder

1. Pull the silicone sensor sleeve over the sensor and stretch it out along the cable.
2. Slip one edge of the sensor into the holder and gently pull the cable through the base. **[A]** Be sure the flat surface of the sensor faces the gripping edges and the back (ridged) surface sits in the "window" of the holder.
3. Snap the second edge of the holder over the bite piece. **[C]**
4. Slide the holder into the bite piece. **[C]**
5. Attach arm and ring.

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### Cleaning and Sterilization

1. Disassemble instrument(s).
  2. Remove debris from components with hot water and soap.
  3. Bag the components before autoclaving. Do not put plastic pieces in the same bag as metal parts.
  4. Autoclave and cycle per autoclave manufacturer's instructions. Do not exceed 275°F (135°C).
- DO NOT use phenol-based glutaraldehyde.
  - DO NOT use special application cleaners.
  - DO NOT clean with chemicals.
  - DO NOT CHEMICLAVE or DRY-HEAT STERILIZE.
  - DO NOT rest plastic pieces against the autoclave wall.
- Note: Plastic parts have a limited life and will show some discoloration when sterilized by any method.

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DENTSPLY International, Inc.